Cod.
3325

SETTORE DENTALE
Via Don L. Sturzo,6 – 42020 Barco (RE) (ITALY)
Tel. 0522/875159 – 875195 Fax 0522/875579 – 875736
e-mail: reverberi_a@libero.it

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Modello **CLEANER 5**

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(GB) WIRING

It is necessary to connect the plug with a socket provided with earth, making sure that the tension corresponds to that indicated on the back of the machine.

WATER FEEDING

This machine does not need any linking to the water grid; it is fitted up with a boiler, which must be filled up manually. In order to fill up the machine with the water it is necessary to unscrew (in counterclockwise way) the security valve (1), then fill up the boiler using the supplied funnel.

Pour the fluid to reach the maximum capacity (it is possible to pour warm water into the boiler to make the heating of the fluid easier).

It is possible to check the level of the water through the special glass (8). At the end of this operation screw the safety valve and make sure to have not poured any fluid on the machine.

WORKING OF THE MACHINE

A) Press the general switch (7). The pilot of the switch lights up and at the same time the heating of the fluid starts. The pilot (14) showing the working of the element lights up.

It is necessary to wait for a heating time of about 12 minutes (when the fluid is cold). When the heating pilot (14) lights up the machine is in the conditions to start working.

It is possible to check the internal pression of the boiler using the manometer (5). B) Press the special push-button (6) to predispose the working of the gun; the lamp showing the level glass will light up (8).

The steam comes out simply by pressing the button set upon the gun. It is possible to regulate the steam quantity by using the knob (3).

OPERATIONS TO EXECUTE AT THE END OF THE WORK

In order to turn the machine off it is necessary to disconnect the general switch (7) and the gun connection switch (6). It is advisable to loosen the security valve (1) to avoid that the machine remains in pression without any reason (but remember to screw the plug again when you want to reutilize the machine).

IMPORTANT

Never open the security valve (1) and the drain cock (11) when the boiler is in pression.

In order to execute the topping-up of the water it is necessary to switch the machine off and loosen the security valve (1) slightly, in order to let the steam go out. Let the steam go out using the iron; when the manometer (5) shows about 1 bar, slightly loosen the valve (1) until the pression is exhausted. In this way the operator will run no risks. Never leave the machine feeded; in this case switch both the boiler and the gun off.

MAINTENANCE AND CONTROL

- Security valve (1): check periodically that the small hole on the side is free from fur, because it must allow the discharge of the steam in case of anomalous working of the machine.

- Steam tubes and wirings: check the condition of the electric and pneumatic plans periodically, replacing the damaged components if necessary.

- Boiler: periodically discharge the water inside the boiler in order to eliminate the calcareous deposits; to execute this operation properly it is advisable to switch the machine off with the inside fluid cold (with the manometer (5) showing zero), to avoid burns.

Open the drain cock (11) to clear possible deposits away. At the end of this operation close the drain cock carefully.

- **Don't utilize the machiner for other use, from those for that has been realized**
- **Alwais wear DPI in the respect of the disposition antiaccident.**
- **Don't approach to machine with inflammable materials.**
- **Never leave the boiler, with lighted and placed on covering's cloth.**
- **The use is allow only to autorized person and howener adults.**
- **Proibition to clean or verificate during the movement.**
- **Prohibition to use water for putting the fire on eletrric equipement under tension.**
- **Don't put on the machine material with dimension ad weight such as to compromise the stability's machine.**
- **Don't licht the machine in presence of more people that caused accidental contact, could compromise the safety condition on ironing machine.**
- **Don't put in function the device when the cable is not perfectly entre or the structure is damage.**

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(GB) DAMAGE TO THE MACHINE

ATTENTION! In case of bad working, switch immediately the machine off and contact the servicing; show the present handbook to the operator who must execute the repairing.
Some examples of verifiable damages and the relative remedies are listed here below.

THE STEAM DOES NOT COME OUT EVEN IF THE BOILER IS IN PRESSION

- A) Gun connection switch is disconnected:
 - Press the switch (6).

- B) The handwheel for the steam adjusting (3) is closed:
 - Unscrew this handwheel in counterclockwise way.

- C) The cable of the gun is damaged:
 - Replace the cable checking the connections on the clamp.

- D) The coil of the electrovalve is burned (2):
 - Replace the coil and check the working of the valve.

- E) The push-button of the gun is defected:
 - Replace the push-button checking the connexions.

THE BOILER DOES NOT GO IN PRESSION EVEN IF THE GENERAL SWITCH (7) IS CONNECTED

- A) There is no water inside the boiler:
 - Switch the machine off, make sure that there is no pression, unscrew the plug and execute the filling.

- B) The element is burned or it does not work properly:
 - Check the element and replace it if necessary.

- C) The pressure switch does not work properly:
 - Check the tube of the pressure switch, which must be perfectly free and replace the pressure switch if necessary.

STEAM COMES OUT FROM THE SECURITY VALVE

- A) The gasket of the valve is hardened or cracked:
 - Replace the gasket or the security valve (each 2 years).

- B) The pression is too high:
 - Check the pressure switch and the safety thermostat (we suggest to apply to a specialized center).

- C) The security valve is damaged:
 - Because of the importance of this component we suggest to replace it without hesitation.

NOTE

In course of consultation of the drawing of the esploded view, it is possible to obtain the code number by looking for the reference number (placed near the drawn component) and consulting the column "NR. LEG." of the list.

The gun is contemplate as accessory, so is out of guarantee.

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EN292 - EN414 - EN60204-EN97/23/CEE

(GB)

The models of the **CLEANER 5** series have been examined according to the provisions of the Direction **CEE 89/392** and subsequent modifications.

The examination carried out is publicized by the affixing of the plate shown in the figure with **CE** mark on the machine and by the declaration of conformity accompanying the following booklet.

DECLARATION OF CONFIRMITY

CEE 89/392 Direction and following modifications and supplements:

The company **REVERBERI S.R.L., Via Don Luigi Sturzo 6, 42020 BARCO (Reggio Emilia) ITALY**, declare under our exclusive responsibility that the product **STEAM GENERATOR CLEANER 5-model**, this declaration refers to, complies with the provisions of **MACHINE DIRECTOR 89/392** and subsequent modifications and integrations, as well as the following standard:

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